


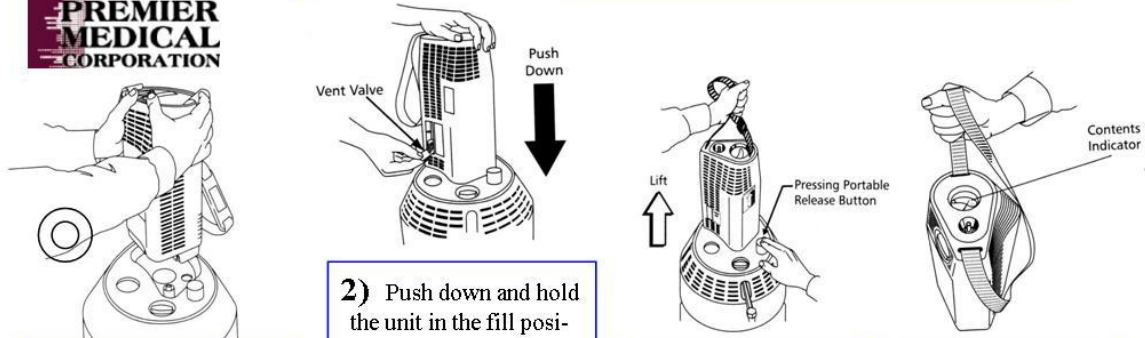

## Liquid Oxygen Fill Instructions



**1)** Dry any moisture from connectors using a clean, dry cloth. Place the portable unit directly over the blue quick connector and press straight down. The portable fits into the cut-out shape on the top of the stationary tank.

**2)** Push down and hold the unit in the fill position. Pull the vent valve lever to the open position. Fill times may vary. When you notice a change in the sound of venting gas followed by a dense, white vapor coming from the cover, push the vent valve lever to the closed position.

**3)** After filling is complete, grasp the straps of the portable and lift straight up while pushing the gray button on the stationary. **DO NOT USE FORCE** while separating the portable.

**4)** To check the oxygen level, hold the strap closest to the contents indicator at least one inch from the unit. Let the portable dangle in the air until the needle has stopped moving.



**24 Hour Service  
(303) 650-4400**



## Liquid Oxygen Flow/Duration

Flow Rate	Portable (Full)	Stationary (Full)
.25	44.0 hours	58 days
.50	27.0 hours	47 days
.75	19.6 hours	31 days
1.0	15.5 hours	23 days
1.5	11.4 hours	15.5 days
2.0	8.5 hours	11.7 days
2.5	6.7 hours	9.4 days
3.0	5.8 hours	7.8 days
3.5	5.0 hours	6.7 days
4.0	4.3 hours	5.8 days
5.0	3.4 hours	4.7 days
6.0	2.6 hours	3.9 days
8.0	2.1 hours	2.9 days
10.0	1.7 hours	2.3 days
15.0	1.1 hours	N/A